

Hidro Health Enzyme

For cleaning all types of soft and gas-permeable contact lenses

This product contains an active substance capable of removing protein and other deposits that accumulate on the lens during daily wear. Enzyme tablets are active against protein, fat, mucin and calcium deposits.

We recommend using enzyme tablets once a week.

Instructions for use with soft lenses

Instructions for general use (multipurpose solutions)

1. Wash and rinse your hands.
2. Fill the lens case with multipurpose solution.
3. Add one enzyme tablet.
4. Insert the lenses in their respective baskets.
5. Keep lenses in the solution overnight.
6. Before inserting your lenses on the next day, rinse them thoroughly with multipurpose solution.

Instructions for use with one-step peroxide system

Wash and rinse your hands. Fill the lens case with the peroxide system disinfecting solution. Add one enzyme tablet at the same time as the neutralising tablet and follow the instructions.

Before inserting your lenses on the next day, it is important to rinse them thoroughly with saline solution.

Instructions for use with gas-permeable lenses

Follow the instructions for general use using multipurpose or conditioning solution for gas-permeable lenses.

Before inserting your lenses on the next day, rinse them thoroughly with the same solution.

Precautions

- Store below 35 °C and out of the reach of children.
- Don't forget to rinse the lenses thoroughly after using enzyme tablets.
- Do not swallow the tablets.
- In case of persistent irritation, stop using the product and consult your contact lens specialist.
- Do not use this product if you are allergic to any of its ingredients.

Composition

Sodium chloride, disodium phosphate, subtilisin 1.5 mg.

